



SUMMIT HILL LABORATORIES

Tinton Falls Business Center
One Sheila Drive
Tinton Falls, New Jersey 07724

VETROSON® LOW-SPEED HANDPIECE

E-MAIL: SALES@SUMMITHILLLABORATORIES.COM


WWW.SUMMITHILLLABORATORIES.COM

Phone: (732) 933-0800

Fax: (732) 933-0055

LOW SPEED HANDPIECE

1. Affix a prophyl angle (polishing) or latch type contra angle (drilling) as follows.
 - a. Hold the handpiece in your left hand, depress the Collar by gently pressing it toward the body of the handpiece and twist ¼- turn.
 - b. Place the attachment fully over the chuck housing.
 - c. Return the ring to its original, “locked” position by twisting ¼- turn counter-clockwise.

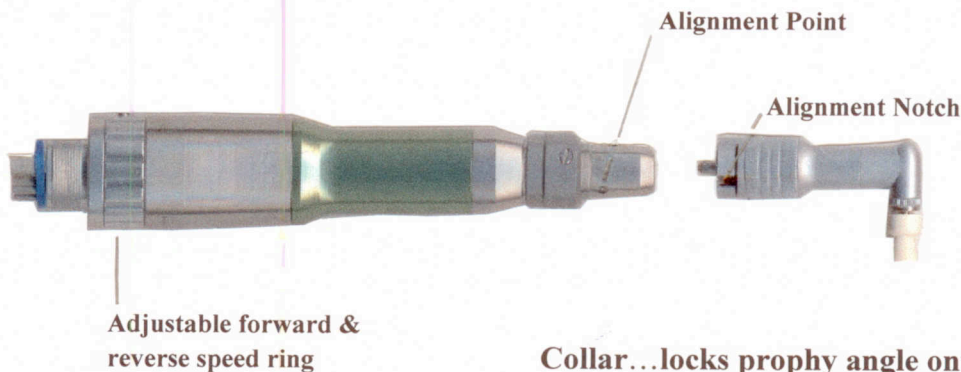

Collar...locks prophyl angle onto Low Speed Handpiece

- **Clockwise....locks**
- **Counter Clockwise....releases**

End view of Handpiece



Lubricate Handpieces

**Optimum....1 to 2 drops of VETROSON®
Lubricant Solution into air
inlet hole after each use.**

Minimum.....1 to 2 drops daily